

# EZee Life™

## Hercules Folding Power Wheelchair Lifter

Model: CH5060



EZee Life™ Products

34 Futurity Gate, Unit 15  
Concord, Ontario L4K 1S6

Prior to the first use of this product, please read this guide carefully and follow the instructions.

## Specifications

Folded Size:	70 x 40 x 23 cm (27.5" x 15.75" x 9")
Unfolded size:	178 x 85 x 70 cm (70" x 33.5" x 27.5")
Weight:	10 KG (22 lbs)
Weight Capacity:	30 KG (66 lbs)
Lifting Height:	1.5 M (60")
Motor:	24 VDC 100 W
Batteries (2):	24 VDC 9 AH
Charger:	24 VDC 2A

## Structure


The lift begins in a compact shape that easily unfolds and telescopes out to its full size. It consists of 4 major components.

1: Head

2: Telescopic ladder style rack

3: Support frame

4: Wheels

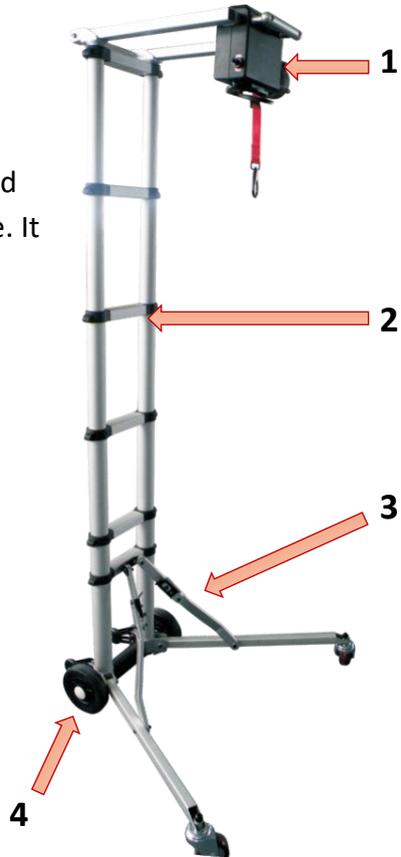

## Operation:


- 1) When unfolding the lift unfold the support frame first so the lift is supported on the ground on the wheels and casters.



## Notes:

- A. The ground should be flat and level where the lift is used.
  - B. Both sides of the support frame should be even and correctly unfolded to prevent imbalance of the lift.
- 2) Pull the circular tubes out one by one from the bottom and fully unfold them into the correct position.




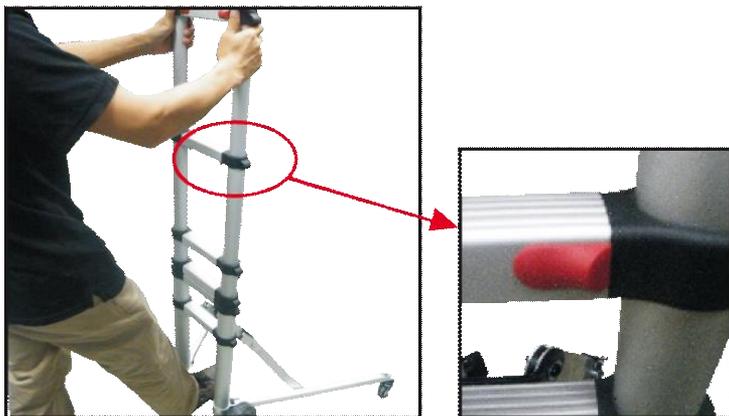


**Note:**

When pulling out each circular tube ensure they are fully extended and the edge of the push button overlaps slightly. If the sections aren't locked in position the lift may fail or be damaged.


- 3) Ensure the maximum weight lifted does not exceed the maximum of 30 KGs (66 lbs). **Overloaded lifts will damage the motor. reduce the lifespan and void the warranty.**

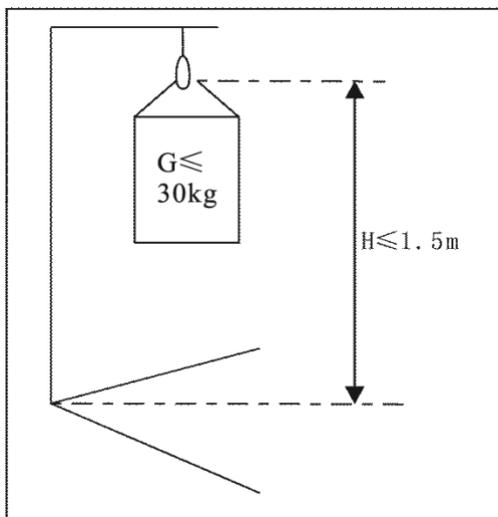

- 4) Lifting strap must be wound around the cylinder a minimum of 3 times to prevent accidents or damage.
- 5) Electrical Connection: Insert one end of the power cable into the socket of the lift and the other end into the scooter/wheelchair charged socket.

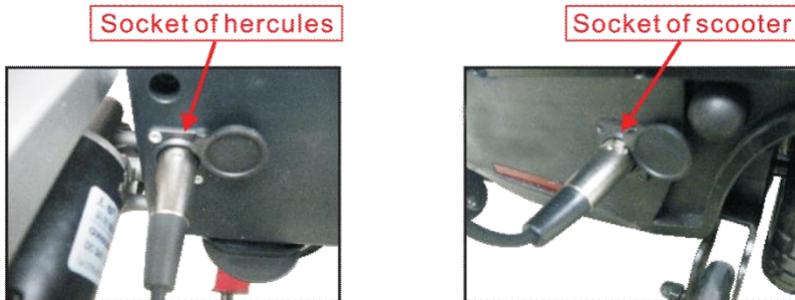


- 6) With the power cord connected move the control switch up to lift the wheelchair and move it down to lower the wheelchair.

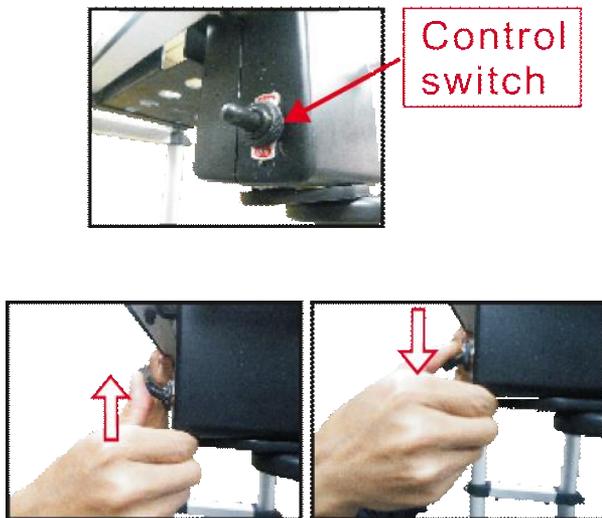

- 7) Height Adjustment: Adjust the height by pulling the circular telescopic tube sections out to their locking position. Only pull out as many sections as needed to lift your wheelchair into your vehicle.



- 8) Push the locking buttons inward to unlock the sections and collapse them one at a time until all sections are collapsed.

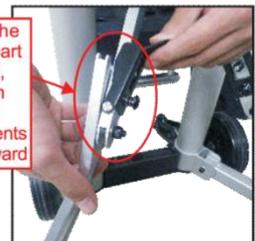

## Note:

- a. When collapsing the sections ensure your fingers are not between the crossbars to prevent pinching.
- b. Ensure the locking push buttons are in the correct position and do not force the collapse or folding of components if they do not move freely.
- c. Do not use the lift if any part is damaged or doesn't seem to be working properly. Contact your dealer for advice or assistance.

## Warranty:

EZee Life™ products carry a one-year warranty against manufacturing defects, faulty materials and workmanship.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the item. Please contact your supplier/dealer for assistance should a defect become apparent.

The warranty does not extend to consequential costs resulting from the fault or defect in the product, in particular freight and travel costs, loss of earnings, expenses or other expenses which may occur.

The warranty does not cover normal wear and tear resulting from normal use or damage caused by accident, misuse, negligence or improper installation or assembly.

The vendor/manufacturer will not accept responsibility for any damages or injury caused by misuse, negligence or non-observance of the instructions supplied.



**Imported & Distributed by:**

HPU Medical Wholesale  
34 Futurity Gate, Unit # 15  
Concord Ontario L4K 1S6  
Local: 416-739-1267  
Toll Free: 1-888-634-5808